










SLEEP SURFACES: INDICATIONS FOR USE



(updated 11.30.2016)

TREATMENT LEVEL	NAME	PHOTO IMAGE	FEATURES	WEIGHT LIMIT
<p>LOW RISK Braden Score 15-18</p>	<p>FOAM MATTRESS (NON-POWERED)</p>	<div style="text-align: center;">  <p>Found on Brown 850 bed frames</p> </div>	<ul style="list-style-type: none"> • Low risk ambulatory patient. • Multi-density foam for Group I pressure redistribution for pressure injury prevention. • At risk for “bottoming out” if patient’s weight distribution is in the mid or lower body section. 	<p>300 lbs (Therapeutic)</p>
<p>MODERATE RISK Braden Score 13-14</p>	<p>ZONE AIRE</p>	<div style="text-align: center;">  <p>Found on varied bed frames: Brown 850 & Advanta</p> </div>	<ul style="list-style-type: none"> • Indicated for prevention and treatment of Stage 1, 2 and 3 - 4 healing pressure injuries on one turning surface. • Automatically adjusts 3 zones within the mattress to patient’s weight distribution + 3 heels zones manually activated by caregiver. • Heel Relief Mode for the patient who is at high risk for pressure ulcer formation on heels. 	<p>300 lbs (Therapeutic)</p>
<p>MODERATE RISK Braden Score 13-14</p>	<p>VERSACARE WITH AIR</p>	<div style="text-align: center;">  </div>	<ul style="list-style-type: none"> • Weight- based pressure redistribution – superior to ZoneAire. • Therapeutic for pressure injury prevention and healing uncomplicated Stage 3 to 4 pressure injuries. • Zoned mattress when bed plugged in. • Integrated scale. • Length of bed can be adjusted 75” – 86”. • 18” from floor to top of mattress. 	<p>500 lbs</p>

<p>MODERATE RISK Braden Score 13-14</p>	<p>ACCUMAX MATTRESS</p>		<ul style="list-style-type: none"> • Prevention & treatment for Stages 1 & 2 and uncomplicated 3 & 4 pressure injuries. • <u>Non-powered</u> dynamic pressure re-distribution. • Can be converted to powered therapy by adding alternating pressure pump, found in SPD or on MAP unit. • Sloped heel section. • 21 inches from top of mattress to floor. 	<p>500 lbs</p>
<p>MODERATE RISK BRADEN SCORE 13-14</p> <p>TO</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>VERSACARE P500</p>		<ul style="list-style-type: none"> • Low air loss therapy for moisture control. • Turn assist feature. • Prevention/treatment of pressure injuries up to Stage 3 or 4. • Egress from chair position at foot of bed. May also exit bed from side as with standard bed. • Motor driven "Intellidrive: for powered transport. • 19" from floor to top of mattress. 	<p>500 lbs</p>
<p>MODERATE RISK BRADEN SCORE 13-14</p> <p>TO</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>STRYKER ISOGEL MATTRESS WITH LOW AIR LOSS PUMP</p>		<ul style="list-style-type: none"> • Integrated scale. • Bed exit alarm system for falls prevention. • Combination mattress of foam and gel. • Indicated for pressure injuries Stages 1-4, Unstageable, and Deep Tissue Injury. • Powered Pump at the foot of the bed will convert the bed to Low Air Loss (NOTE: Leave the pump turned on for maximum benefit to the patient). • Extension available to lengthen the bed and is kept in the Traction Room (0-351). Every nurses' station has a key available for access. 	<p>500 lbs</p>

<p>MODERATE RISK BRADEN SCORE 13-14</p> <p>TO</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>ENVISION MATTRESS</p>		<ul style="list-style-type: none"> • Rental Mattress: Call SPD. • Low air loss therapy for excessive moisture control. • Step-down surface of choice from Clinitron Rite Hite. • Indicated for pressure injuries Stages 1-4, Unstageable, and Deep Tissue Injury. • Turn assist feature. • Max Inflate feature to decrease shear and drag during patient transfers. 	<p>400 LBS (Therapeutic)</p>
<p>MODERATE RISK BRADEN SCORE 13-14</p> <p>TO</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>TOTAL CARE SPORT</p> <p>RESPIRATORY THERAPY SYSTEM</p>		<ul style="list-style-type: none"> • Used for prevention and treatment of pulmonary and skin complications related to immobility. • 30 degree head of bed alarm and trending. • Turn assist feature. • Continuous lateral rotation therapy, percussion and vibration. • Full chair egress or may exit bed from side as with standard bed. • Adjustable length. • Integrated scale. • Motor driven "Intellidrive: for powered transport. 	<p>460 lbs</p>
<p>MODERATE RISK BRADEN SCORE 13-14</p> <p>TO</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>STRYKER IN TOUCH</p>		<ul style="list-style-type: none"> • ICU use only. • Integrated bed scale. • Bed exit alarm system. • Used for prevention and treatment of pulmonary and skin complications related to immobility. • Indicated for Stages I-IV, Unstageable, and Deep Tissue Injury. • Integrated "air pod" technology with Low Air Loss therapy. • Lateral rotation feature. • Turn assist feature. 	<p>550 lbs</p>

<p>BARIATRIC</p> <p>LOW RISK BRADEN SCORE 15-18</p>	<p>TOTAL CARE BARIATRIC - FOAM</p>		<ul style="list-style-type: none"> • Foam mattress surface for management of the low risk ambulatory obese patient. • Egress from chair position at foot of bed or may exit bed from side as with standard bed. • Adjustable length. • Integrated scale. • Does <u>not</u> accommodate overhead frame and trapeze. • Motor driven “Intellidrive” for powered transport. 	<p>460 lbs</p>
<p>BARIATRIC</p> <p>MODERATE-HIGH RISK BRADEN SCORE 13-14</p>	<p>TOTAL CARE BARIATRIC PLUS</p>		<ul style="list-style-type: none"> • Low Air Loss Therapy for moisture control. • Egress from chair position at foot of bed or may exit bed from side as with standard bed. • Seat-deflate option for low bed height for easier. • 40” wide turning surface. • Adjustable length. • Turn assist feature. • Motor driven “Intellidrive” for powered transport. • Integrated bed scale. 	<p>500 lbs</p>
<p>BARIATRIC</p> <p>LOW RISK BRADEN SCORE 15-18</p>	<p>EXCEL BARIATRIC BED WITH FOAM MATTRESS</p>		<ul style="list-style-type: none"> • Bariatric Foam Mattress. • Integrated scale. • Not motorized. • Adjustable length. • Bed width can expand from 40” – 50” to accommodate patients with wider girth. • Seat and foot deflate for low bed height. • Does NOT sit in chair position. Patient egress as with standard bed. • Patient Helper (trapeze) specific to bed. 	<p>1000 LBS</p>

<p>BARIATRIC</p> <p>MODERATE RISK BRADEN SCORE 13-14</p> <p>HIGH RISK BRADEN SCORE 10-12</p>	<p>EXCEL BARIATRIC BED WITH AIR</p>	 <p>A white hospital bed with a red mattress and a white overhead lift arm with a red handle. The bed has side rails and a control panel on the right side.</p>	<ul style="list-style-type: none"> • Rental Bed. Call SPD. • Low Air Loss therapy for moisture control. • Zoned pressure redistribution. • Same adjustable frame features as Excel Foam bed. • Integrated bed scale. 	<p>1000 LBS</p>
<p>VERY HIGH RISK BRADEN SCORE 9 OR LESS</p>	<p>CLINITRON RITE HITE</p>	 <p>A white hospital bed with an orange mattress and a white overhead lift arm. The bed has side rails and a control panel on the right side.</p>	<ul style="list-style-type: none"> • Rental Bed. Call SPD. • Combination therapies: Low Air Loss for moisture control & Air Fluidized for maximum pressure redistribution through immersion. • Treatment of Stage 3 or 4 pressure injuries, flaps, grafts, burns and lower leg/trunk wounds, excessive maceration, intractable pain. • HOB can be elevated. • Patient Helper (trapeze) available. • Easier exits and transfers. • No scale. • No Trendelenburg or reverse Trendelenburg positions. 	<p>350 lbs</p>